




Oasis[®] Bed Bathing Washcloths are impregnated with a unique combination of skin cleansers to help remove sweat, dirt, urine and faeces from the skin. The large, soft washcloths are intended to be used individually, with one single-use cloth per anatomical region.

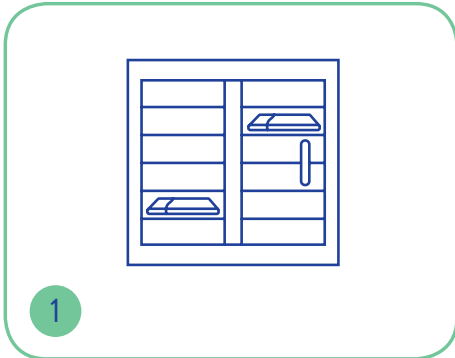


1 Remove wipes/washmitts from the dedicated Oasis Warmer

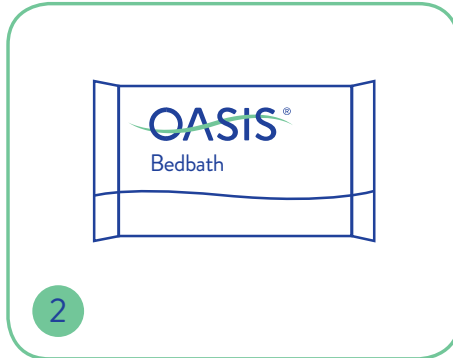


2 Or use at room temperature

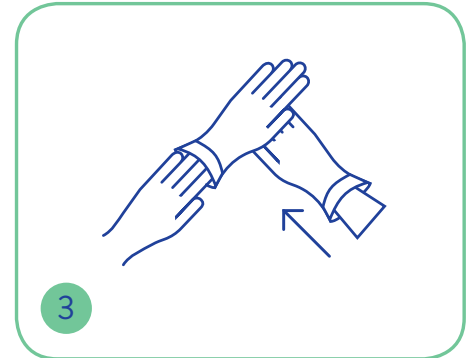


3 Apply protective gloves

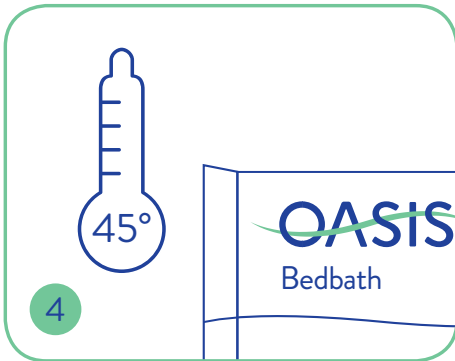


4 Note how many wipes/washmitts are in each pack and check temperature

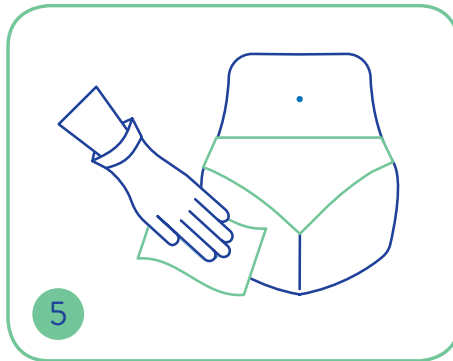

5 Use one wipe/washmitt to per anatomical region

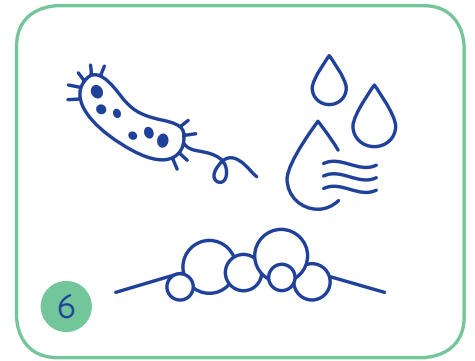

6 The wipes/washmitts will cleanse and moisturise the skin

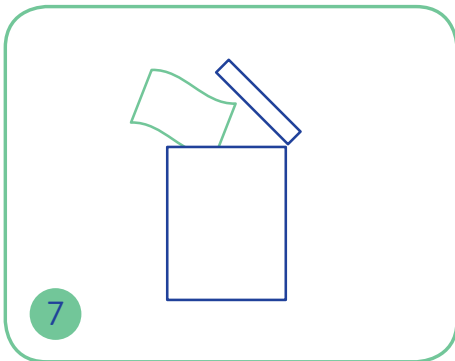

7 Dispose of used wipes/washmitt

Description	Size	Pack Qty	Case Qty	Code
Oasis [®] Bed Bath Scented	33 x 22cm	10	24	RSC901
Oasis [®] Bed Bath Unscented	33 x 22cm	10	24	RSC902N
Oasis [®] Bed Bath Scented	33 x 22cm	8	30	RSC912
Oasis [®] Bed Bath Unscented	33 x 22cm	8	30	RSC913N
Oasis [®] Washmitt Scented	-	8	30	RSC604A
Oasis [®] Washmitt Unscented	-	8	30	RSC605A



Perfumed and unperfumed options available