

3 Things to Consider When Choosing your Surface Wipes



Do I need to clean the surface?

Detergent wipes

Detergents, whose main purpose is to **release dirt from a surface**, are essential to the cleaning process.

Azo™ Detergent wipes offer a convenient and effective alternative to traditional cleaning, specifically designed to remove dirt and organic matter from surfaces. Detergent wipes have been shown to physically **remove over 99.9% of surface contamination** without the need for any harsh chemicals or disinfectants.

Did you know?



Flushable and Maceratable available!

Do I need to disinfect the surface?

Alcohol wipes

Alcohol is a widely-used disinfectant, effective against pathogens which can cause infection.

Azo™ 70% IPA wipes offer a **powerful quick-drying disinfectant** with fast kill rates, effective against MRSA, E.coli and a range of other pathogens including **Coronavirus**.

Did you know?



CE marked options are also available. These are red!

Do I need to do both?

Universal wipes

Two-in-one surface cleaning and disinfection wipes are convenient as they are a combination of detergent and disinfectant.

Azo™ Universal combines proven effective biocides with a powerful detergent to **effectively clean and disinfect surfaces**, offering broad spectrum efficacy against a range of bacteria, yeasts, fungi and viruses including **Norovirus** and **Coronavirus**.

Did you know?



Available in a variety of formats, including smaller flow-wrap packs.

How To Use Each Product

azo™

Good cleaning and disinfection practices will help ensure the risk of cross-contamination is reduced and that maximum product efficacy levels are achieved. See below guide to using Azo™ wipes and sprays.

1.



Observe and select correct PPE and product for specific requirement.

2.

If using an Azo™ wipe, remove wipe from packaging and ensure the pack is resealed to avoid wipes drying out.



3.



If using an Azo™ spray, apply directly to the surface and wipe with a clean cloth.

4.



Wipe over the surface working from clean to dirty. Do not go over the same area twice with the same wipe.

5. Ensure surface remains wet for target contact time.



6.



Once used, discard appropriately. Only clearly labelled wipes can be disposed of in a macerator or flushed down the toilet.

Surface Compatibility

Azo™ wipes and sprays are compatible with most non-porous surfaces and equipment found in healthcare, scientific and hospitality sectors.

For sensitive equipment or uncommon surfaces, we would recommend contacting the manufacturer of these materials for guidance.

