



# Surface Cleaning & Disinfection Best Practice

Surface cleaning & disinfection is one of the most effective ways to stop the spread of infection. Various studies have shown that contaminated surfaces are an established route of transmission for high-risk pathogens.<sup>1</sup>


See below best practice guide when using AZO range, helping to ensure effective surface cleaning and disinfection:

1.



Observe the correct PPE and select the appropriate product for your specific requirements.

2.

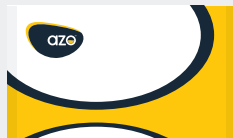

Dispense wipe(s) from the pack, ensuring the pack is resealed to prevent wipes drying out.

3.




If using a spray, apply directly to the surface before wiping with a clean cloth.

4.




Wipe the surface, working from clean to dirty. Do not go over the same area twice with the same wipe.

5.

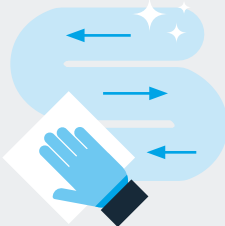

Always work towards yourself in a fluid 'S' shaped motion.

6.

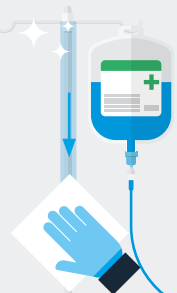

For larger surfaces/equipment, work from top to bottom.

7.



Ensure the surface remains wet for the required contact time and allow to air dry.

8.



Once used, discard the wipes appropriately. Only clearly labelled wipes can be disposed of in a macerator or flushed down the toilet.

**Please note that the above is intended as best practice guidance only. Ensure you refer to your local infection prevention policy and procedures, as required.**